color

Oral swab collection kit

Please read all instructions before swabbing.

Read first

- Read all instructions carefully.
- Wash your hands before collecting your sample.
- Do not eat, drink, smoke, or brush your teeth for 30 minutes before swabbing your throat.
- Do not drink alcohol on the day of collection.
- The collection tube contains liquid. **DO NOT EMPTY THE LIQUID FROM THE COLLECTION TUBE.**

This kit includes:


1 collection tube

1 oral swab


1 biohazard bag
(with absorbent pad)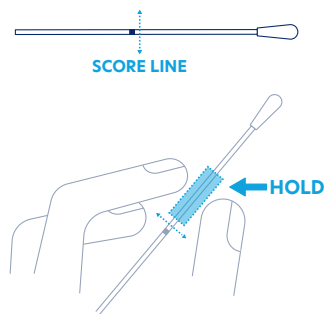


1

Take the swab out of its packaging. Hold the swab between the faint score line and the cotton-tipped part of the swab.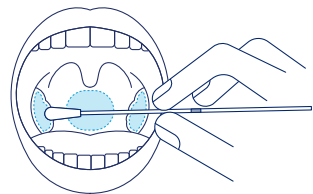


2

Put the swab into your mouth and rub the swab tip against the sides and back of your throat for 10 seconds. Don't touch your cheeks or tongue. Carefully take out the swab.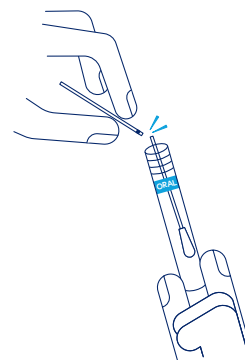


3

Find the collection tube marked with a blue "ORAL" label. Put the swab in the collection tube with the soft tip facing down. Break the swab by bending it against the collection tube. The end of the swab will snap off. Discard the end of the swab.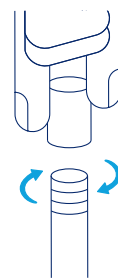

4

Twist the lid tightly shut on the collection tube. Do not break the foil top of the tube.

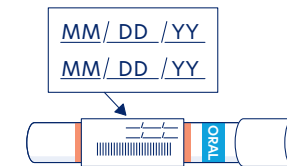5

Write your date of birth and the collection date (today's date) in MM/DD/YY format onto the tube where noted.

6

Put the tube into an empty biohazard bag. Do not remove the absorbent pad that's in the biohazard bag. Make sure the seal of the bag is closed completely.

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