color

Urine collection kit

Please read all instructions before collecting your urine.


Read first

- Read all instructions carefully.
- Wash your hands before collecting your sample.
- Do not drink alcohol on the day of urine collection.
- Do not collect urine during your period or within 24 hours of intercourse.
- The collection tube contains liquid. **DO NOT EMPTY THE LIQUID FROM THE COLLECTION TUBE.**

| This kit includes:

1 collection tube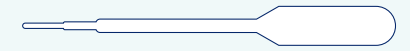

1 pipette (dropper)

1 collection cup

1 biohazard bag (with absorbent pad)


1

Do not pee for at least 1 hour before collection. It's best to collect your urine first thing in the morning.


2

Pee directly into the collection cup. It should be almost half filled.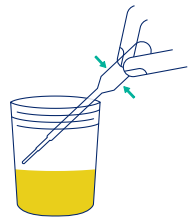


3

Use the pipette (dropper) to transfer your urine from the collection cup into the collection tube marked with a green "URINE" label.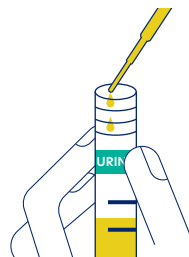


4

Fill the tube until the combined liquid is between the two black lines. Do not fill the tube past the maximum fill line. The lab cannot accept the tube if it's too full.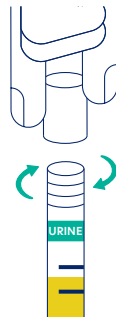

5

Twist the lid tightly shut on the collection tube. Do not break the foil top of the tube.

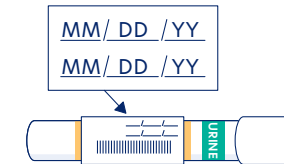6

Write your date of birth and the collection date (today's date) in MM/DD/YY format onto the tube where noted.

7

Put the tube into an empty biohazard bag. Do not remove the absorbent pad that's in the biohazard bag. Make sure the seal of the bag is closed completely.

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