

## Monitoring and maintaining program

### Access dashboards

If you are a district director or school lead, you'll receive access from Color to two real-time dashboards to manage the testing of your students and staff.

### Communicate results

Your district director will have decided if testing results will be sent to staff, students, and their parents/guardians through Color. We recommend that you follow up with positives to communicate next steps per your district's guidelines and any state or local recommendations.

### Order materials

Additional starter packs (like this one), test kits, and ship-back materials can be purchased at [fishersci.com](https://fishersci.com). Please order five or more days in advance of testing to ensure the materials arrive in time.

### Get support

Color is here to support you and your team. For more information and details on the above, please visit [color.com/readycheckgo-support](https://color.com/readycheckgo-support). You'll find a form to reach support, and details on Zoom office hours where you can ask Color staff questions in real time. If you are having an urgent issue, you can reach us at 1-844-658-0387.

Learn more about testing at [thermofisher.com/readycheckgo](https://thermofisher.com/readycheckgo)


Order test kits and ship back materials at [fishersci.com](https://fishersci.com)

Find detailed instructions on running your testing program at [color.com/readycheckgo-training](https://color.com/readycheckgo-training)

Find email templates for communicating the program with your school community at [color.com/readycheckgo-resources](https://color.com/readycheckgo-resources)

**ThermoFisher**  
SCIENTIFIC

**color**

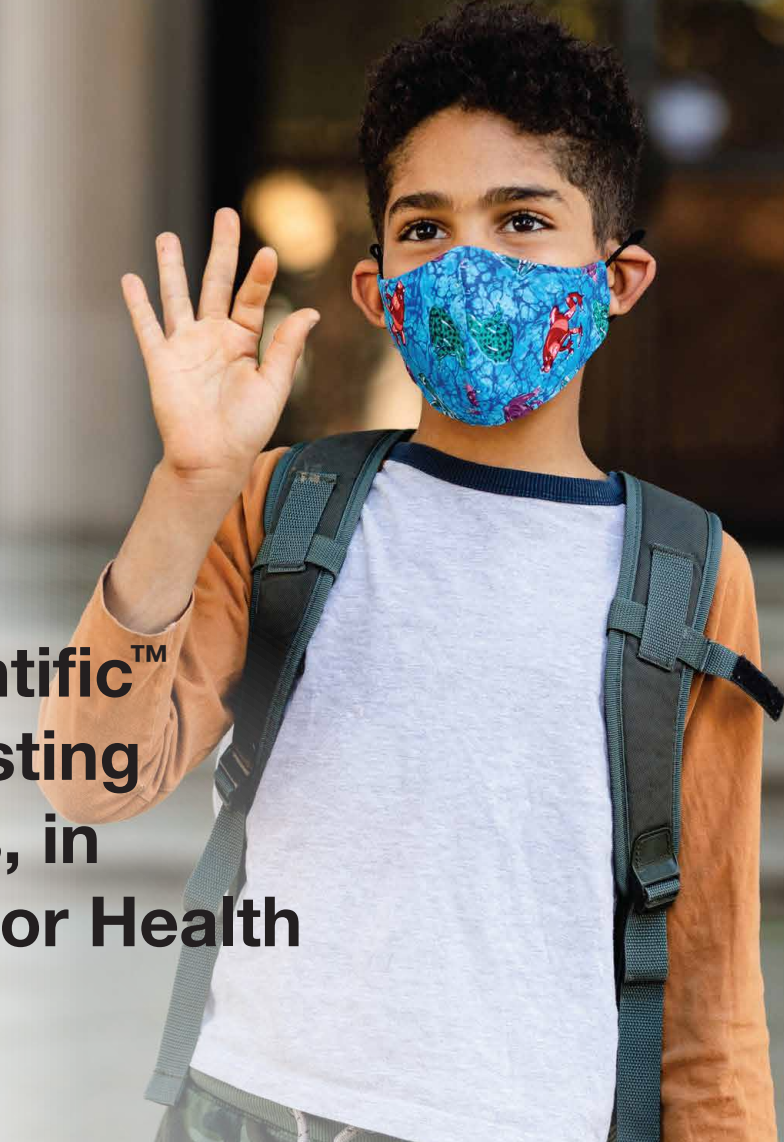

**Welcome to the  
Thermo Fisher Scientific™  
ReadyCheckGo™ Testing  
Program for schools, in  
partnership with Color Health**

**Inside this box, you will find instructions  
and materials for your testing stations.**

## Program overview

Here's an overview of the program and the steps you need to take to run a testing program. For more detailed instructions, head to [color.com/readycheckgo-training](https://color.com/readycheckgo-training).

### Onboarding and site setup

- Identify testing team
- Sign up for Color accounts
- Order materials
- Set up testing stations

### Participant registration

- Engage your community
- Register participants
- Collect consent

### Collecting and shipping samples

- Collect samples
- Pack and ship samples (if applicable)

### Monitoring and maintaining program

- Access dashboards
- Communicate results
- Order supplies
- Get support

For additional resources, detailed instructions, and FAQs, please visit our website at [color.com/readycheckgo-training](https://color.com/readycheckgo-training)

## Onboarding and training

### Identify your testing team

It takes a team to run an effective testing program, and we recommend selecting people from across your organization for roles we've outlined on [color.com/readycheckgo-training](https://color.com/readycheckgo-training).

### Create Color accounts

Each staff member, including the district director, school leads, and testing administrators, must create a Color account.

### Obtain materials

You'll need to order test kits and return shipping materials. Plus, we suggest hand sanitizer, waste bins, and personal protective equipment. Get all of this and more at [fishersci.com](https://fishersci.com).

### Set up testing stations

Testing stations are where participants provide their samples. There can be multiple testing stations at each collection site, like in individual classrooms, or one testing site in a central location, like the cafeteria.

## Participant registration

### Engage your community

You can find email templates for communicating this program with your school community at [color.com/readycheckgo-resources](https://color.com/readycheckgo-resources). We suggest sending your school's unique registration link to testing participants. Color will email school leads their registration links.

### Register participants

All testing participants must register for Color themselves or with a parent or guardian. They'll indicate their contact information and answer questions required by federal and state authorities. We recommend that you provide access to a computer or tablet that participants can use to register on-site if they are not able to do so in advance.

### Collect parent or guardian consent

During the online registration process, participants, or a parent or guardian of a participant under the age of 18, must provide one-time consent and HIPAA authorization to share data securely with your school.

## Sample collection

### Collect samples

Head to [color.com/readycheckgo-training](https://color.com/readycheckgo-training) for instructions for sample collection according to the type of testing your program uses.